**AUSTRIA**

Bauerfeind Ges.m.b.H.  
Hainburger Straße 33  
1030 Wien  
P +43 (0) 800 4430-130  
P +43 (0) 800 4430-131  
E info@bauerfeind.at

**BENELUX**

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 english

## Dear Customer,

Thank you for choosing a product from Bauerfeind. The SecuTec Genu knee orthosis<sup>1</sup> is a high-quality medical product.

Please read through these **instructions for use** carefully and if you have any questions, contact your physician, medical retailer or our Technical Service department.

## Intended use

The SecuTec Genu orthosis is an approved therapeutic aid for supporting your treatment. It secures your knee during therapy and can be used in conjunction with physiotherapy to help train your muscles.

 The SecuTec Genu must be individually adjusted for the patient by a trained specialist<sup>2</sup>/orthotist in order to ensure maximum effectiveness and optimum wearing comfort.

## Indications

- Anterior and/or posterior instability due to a cruciate ligament tear (ACL/PCL)
- After ligament surgery/syndesmoplasty
- Complex instabilities (traumatic/degenerative)
- To restrict the range of movement of the knee (e.g. after meniscal suturing or meniscal replacement)
- In the conservative, functional treatment of collateral ligament injuries

## Risks of using this product

### Important information

- The SecuTec Genu functional orthosis is a prescription product that should be worn as directed by a physician. The SecuTec Genu must be worn according to the specifications in these instructions and for the listed indications.
- Product liability cannot be assumed in the event of inappropriate use.
- No improper modifications may be made to the product. Failure to comply with this requirement may adversely affect product performance, thereby excluding any product liability.
- Should symptoms increase or should you notice any unusual changes, please contact your physician without delay.
- Combining the support with other products (e.g. compression stockings) must be discussed beforehand with your physician.
- Avoid contact with ointments, lotions, or any substances containing grease and acids.
- Should any problems with the fit of the orthosis arise, for instance if the shape of your leg changes (e.g. due to an increase or reduction in muscle), or should the product have a functional defect, please immediately contact your medical supply retailer.
- No side effects that affect the entire body have been reported to date. This assumes the orthosis is used/fitted correctly. Any medical aids – supports and orthoses – applied externally to the body can, if tightened excessively, lead to local pressure symptoms or, rarely, constrict blood vessels or nerves.

 Contains natural rubber latex (elastodiene) which can cause skin irritation in persons with an allergy. Take off the orthosis if you notice any allergic reaction.

## Contraindications

No clinically significant hypersensitive reactions have been reported to date. If you have any of the following conditions, such aids should only be fitted and worn after consultation with your physician:

1. Skin disorders/injuries in the relevant part of the body, particularly if inflammation is present. Likewise, any raised scars with swelling, redness, and excessive heat build-up
2. Varicose veins (varicosis)
3. Impaired sensation and circulatory disorders of the legs/feet (e.g. diabetes mellitus).
4. Impaired lymph drainage – including soft tissue swellings of uncertain origin located away from the fitted aid.

## Application instructions

### Size determination

Please determine the correct size in accordance with the specifications.

### Procedure for the first fitting of the SecuTec Genu orthosis

#### Preparation (only by a trained specialist<sup>2</sup>)

- Mark the compromise axis of rotation of the knee joint (as defined by Nietert) at a 30° bend.
- The pivot point of the orthosis hinges is approximately 4 mm above an imaginary line in the center of the hinge cover.

#### Adjustment options for the hinge (to be performed by a trained specialist only)

- The hinge cover can be opened via a screw fastening. This also serves to fasten the movement-limiting wedges (extension – front/flexion – rear) and protects the hinge mechanism against dirt.
- Opening the screw fastening (quarter turn until the arrow points at the open lock symbol) allows the hinge protection to be removed and the desired stops to be changed.
- The desired extension and flexion wedges are selected from the enclosed frame and are used. The following limitations are possible: Extension: 0°, 10°, 20°, 30°, 45° Flexion: 0°, 10°, 20°, 30°, 45°, 60°, 75°, 90°
- Then, click the hinge cover onto the hinge, check whether it is correctly positioned and firmly attached to the hinge, and close the screw fastening.

**Static fitting (only by a trained specialist)**

- Perform the initial adjustment with the straps open.
- Bend the knee to be treated to a 30° angle and place the SecuTec Genu on it so that the hinge pivot point of the orthosis hinge is aligned with the compromise axis of rotation as defined by Nietert.
- Check the fit of the frame parts when they are laid flat to make sure their circumference and longitudinal direction are correct and adjust them if necessary.
- A bending iron with protective caps should be used for any extensive bending work.
- Select the required thickness of knee joint cushions and insert them (there are two thicknesses to choose from).
- If necessary, the shin cushion provided can be attached in order to redistribute pressure in the event of pressure pain in the shin region or swelling around the tibial tubercle (Tuberositas tibiae).

**Putting on the SecuTec Genu**

- The leg is bent at a gentle angle (approximately 30°). The position of the kneecap is then determined through palpation.
- The center of the hinge (pivot point of the orthosis hinges) is positioned at approximately the same height as the center of the kneecap.
- To treat rear instability (PCL), the (proximal) strap near the hinge can be shifted forwards (ventrally). To treat a complex instability, the enclosed sixth strap (ventral, proximal) can be used.
- Mount all the quick-fit fasteners in the frame. Decide on the calf strap height that provides the optimum fit for the patient. If you mount the quick-fit fasteners for this strap in the upper part of the orthosis, thread the opposite end of the strap out of the frame and push the stopper downwards before re-fastening the Velcro closure on the strap.
- Start by tightening the calf strap, followed by the strap beneath it.
- Then tighten the bottom thigh strap at the back, followed by the strap above it. (The quick-fit fasteners are numbered so that you can easily see the order in which to close the straps.) – When closing the straps, check whether the orthosis has rotated laterally around the leg and correct this if necessary.
- Position the front strap with the protective cushion for the tibial tubercle in such a way that the strap can be closed without exerting any pressure. After all the straps have been closed, the orthosis must exert an even pressure when the muscles are tensed.
- Once the orthosis has been put on, check once more that it is in the correct position.

**Taking off the SecuTec Genu**

- Grip the handle of the quick-fit fastener and unhook it from the thigh frame.
- Then lift the upper orthosis frame off the thigh. During this step, the knee joint cushions stay in contact with the knee joint.
- Now remove the straps from the lower leg frame of the orthosis.
- The top and, if necessary, the bottom strap at the front can remain closed.
- After all the straps have been removed, the orthosis can be taken off the knee joint from the front.

### Cleaning instructions

⚠ **Never expose the orthosis to direct heat (e.g. central heating, sunlight etc.)!**

- The aluminum frames of the orthosis are coated and can be cleaned using pH-neutral soap.

We recommend washing the cushions and straps by hand at 30°C.

This product has been tested by our in-house quality management system. However, should you have any complaints, please contact your local medical supply retailer.

### Where to apply the orthosis

In accordance with the indications (knee). Please see intended use.

### Maintenance instructions

If handled and cared for correctly, the product is practically maintenance-free.

### Assembly and fitting instructions

The SecuTec Genu is supplied as a pre-assembled orthosis in standard sizes. The first fitting, which includes the hinge adjustments and the static fitting, must be carried out by a trained specialist.

### Technical specifications/parameters

The SecuTec Genu is a functional orthosis for the knee joint. It consists of a frame, hinges, straps, quick-fit fasteners and cushions.

### Notes on reuse

The SecuTec Genu is intended for the treatment of one patient only.

### Warranty

The statutory regulations of the country of purchase apply. Where relevant, the country-specific warranty regulations between the buyer and retailer apply. Please contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty.

Please do not make any alterations to the product yourself. This particularly applies to products that have been individually adjusted by a trained specialist. Please follow our instructions for use and care. They are based on years of experience and ensure that our medically effective products continue to work for a long time. After all, only aids that function perfectly will help you. Failure to follow the instructions can also limit the warranty.

### Disposal

- This product can be disposed of according to national laws.
- For information regarding the options available for returning the packaging to ensure it is correctly recycled, please observe the instructions from the sales outlet where you purchased this product.

### Version: 2016-12

<sup>1</sup> Orthosis = orthopedic aid used to stabilize, relieve, immobilize, control, or correct the limbs or the torso

<sup>2</sup> A specialist is any person who is authorized, according to their state’s regulations, to fit and provide instructions on the use of supports/orthoses.

--	--

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